

Easy access



Encore patient management makes it quick and simple to obtain vital patient therapy intelligence.

Knowing exactly what patients are experiencing at any given moment of therapy is key to making intelligent patient management decisions.

Encore patient management contributes to an intelligent sleep therapy system by providing timely and secure access to comprehensive patient therapy data. Through an intelligent web-based platform, EncoreAnywhere delivers on its name, providing the care team - regardless of where its members are located - with instant access to patient data via new wired or wireless modem options. As a result, the team can collaborate and respond with prescription changes in real time. EncorePro provides the same level of data via an SD card. Both systems are intuitive and secure, making it simpler and more practical to access the patient intelligence that contributes to more intelligent patient and business management.

EncoreAnywhere: intelligence at your fingertips

With the new wireless or wired modem connection, EncoreAnywhere automatically downloads data for instant access. Through a secured-access website available via any computer, physicians, sleep lab professionals, and home care providers can view timely, accurate patient information and then instantly change prescriptions or communicate as necessary. Thanks to a highly intuitive interface, putting such intelligence to work is surprisingly simple, efficient and cost effective. Better patient compliance and increased care team efficiency; that's intelligent therapy management.

EncorePro: simple and secure

The next generation of EncorePro is better than ever for day-to-day patient management. We've simplified the interface while beefing up security, making all of the notifications, reminders, contact records, patient data and reports more secure and even easier to access.



	TATA DESCRIPTION OF	-			and the second second second		
	Fraine, 3.dy 27, 2487				as black		
	and the second second second second				Contraction of the other states		
	BA DROVERSTA LINES			And Sectors	104		
	1. initian	0.388	- B.H.	Admyri, Sife Amagin, Br	Contrapped researching stephent		
	A. Stranger	08		hann Broad	Net spin out (and 1995) Net spin out (and 1995)		
	and the last	03		ngerg, Nav	influence of the local in monitorial day		
	12	0		-			Dependence
	200 100 0000		0	Constant of	State Street		and Research 1 Last land man
	2211.012	8			0 0 10		
	- Binnes	8		100/10100101	The last in the standard	And the second second	
	The set of the set			100	Address of Section	ALC: Made No.	California Salaria Antonio
		in the second		The Party of the	committee .	August a	united.
re-Anjin				Al (Transf Registration) Area Al (Al and Al			inter inter order determin talen talen talen
	and and the last		l' illa		3	The second	
Date:	(And Andrew) Pro-	and beauting by		and the local	1. j		
*	And the second second	na an Palatra na m	-		-		
			_				

EncoreAnywhere subscription	1999999
EncoreAnywhere administrator guide - homecare provider	1056401
EncoreAnywhere administrator guide - sleep lab and physician	1062233
Take Ten referral marketing package	1056397
Take Ten referral set-up and training package	1056398
EncorePro 2 software	1054785
EncorePro 2 user guide	1060291
EncorePro 2 installation guide	1054258
SD card reader	Part number 1047300
SD card reader	1047300
SD card reader SD card cover (attached to device)	1047300 1063858
SD card reader SD card cover (attached to device) SD cards (10 pack)	1047300 1063858 1063859
SD card reader SD card cover (attached to device) SD cards (10 pack) SD card mailer (10 pack)	1047300 1063858 1063859 1065146
SD card reader SD card cover (attached to device) SD cards (10 pack) SD card mailer (10 pack) Encore reports manual	1047300 1063858 1063859 1065146 1055272
SD card reader SD card cover (attached to device) SD cards (10 pack) SD card mailer (10 pack) Encore reports manual Sleep therapy modem systems	1047300 1063858 1063859 1065146 1055272 Part number
SD card reader SD card cover (attached to device) SD cards (10 pack) SD card mailer (10 pack) Encore reports manual Sleep therapy modem systems Wired flow modem	1047300 1063858 1063859 1065146 1055272 Part number 100200VV

Respironics, Encore, and EncoreAnywhere are trademarks of Respironics, Inc. and its affiliates.



© 2009 Koninklijke Philips Electronics N.V.All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.

Hoech KW 07/02/09 MCI 4102519 PN 1064785

Philips Healthcare is part of Royal Philips Electronics

Respironics Asia Pacific +65 6298 1088 Respironics Australia +61 (2) 9666 4444 Respironics Europe, Middle East, Africa +33 1 47 52 30 00 Respironics United Kingdom +44 800 1300 845

helps users prioritize patient intervention needs quickly.

An easy-to-navigate My Day page

Prescription changes can be made securely and as swiftly as the push of a button. Updates to the device are automatic.

Summary and daily detail reports of **Therapy** data – including patient flow waveforms – are easily viewed and arranged by a specific time frame, including our "Best 30" report which provides the best 30 consecutive days of greater than four-hour sessions. In addition, internal device memory stores six months of compliance data and five days of patient flow waveforms that can be transferred by the Encore data card or modem at any time.

> Philips Respironics 1010 Murry Ridge Lane Murrysville, PA 15668

Customer Service +1 724 387 4000 800 345 6443 (toll free, US only) www.philips.com/respironics